

haakaa®

SILICONE BREAST PUMP & BOTTLE TOP SET

English

Product Name: Silicone Breast Pump & Bottle Top Set**Materials:** Medical-Grade Silicone, PP, PPSU**Dimensions:** Bottle Body (160ml) - 77 x 77 x 71mm

Bottle Body (250ml) - 77 x 77 x 105mm

Flange - 71 x 71 x 108mm

Attachment Ring - 84 x 79 x 20mm

Nipple - 73 x 73 x 58mm

Cap - 75 x 75 x 50mm

Volume: 5.4 oz./160ml

8 oz./250ml

Colours: Peach / Grey**Contents:**
1 x Gen. 3 Silicone Bottle Body
1 x Silicone Pump Flange
1 x Attachment Ring
1 x Silicone Bottle Anti-Colic Nipple - S/L
1 x Silicone Bottle Replacement Cap**Standard:** EN 14350**Features:**

- Made with medical-grade silicone.
- Gen. 3 Breast Pump allows you to express directly into the bottle without any mess or spills!
- Compatible with all Gen. 3 attachments.
- Super soft silicone pump flange accommodates different breast sizes.
- Bottle nipple features valves to allow your baby to drink continuously without gasping for air or swallowing bubbles.
- Wide base helps prevent accidental spills.
- No cords or batteries required.
- Safe in microwaves, sterilisers and boiling water.
- BPA, PVC and phthalate-free.

Cleaning and Care:

Before first use, disassemble and clean the product before placing non-plastic components in boiling water for 5 mins to sterilise. To sterilise, use a steam steriliser or boil in water for 2-3 minutes. To sterilise plastic components, boil in water for no longer than 2-3 minutes. Wash before and after each use in warm soapy water. Other common methods of heating, cleaning, storage and use not listed in this manual may damage the product. Do not use any bleach-based agents or sterilising tablets to clean these products. Do not use UV sterilisers to clean these products as it may impact their lifespan.

Caution:**For your child's safety and health****WARNING!**

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Throw away at the first signs of damage or weakness.
- Keep components not in use out of the reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding teats as a soother.
- Always use this product with adult supervision.
- Do not store near any sharp objects.
- This product is not a toy. Do not use this product for anything other than its intended purpose.
- Take extra care when microwave heating; localised overheating can occur. Always mix the heated food or liquid and check the temperature before feeding. Do not heat breastmilk in the microwave.
- Inspect the feeding teat before each use by gently pulling it in all directions. Throw away at the first signs of damage or weakness.
- Do not leave feeding teat in direct sunlight or heat, or leave in non-bleach-based sterilising solution for longer than recommended, as this may weaken the product.

NOTE: As silicone can stick to itself, the nipple hole or valve opening may seal. To fix, gently rub the end of the nipple or valve between your fingers until the hole reopens. We recommend replacing the nipple every 3 months.

Volume measurements are an approximation and should be used for general reference only. Always follow your country's guidelines and recommendations to safely express and store breastmilk.